


	IFU (Instruction for Use) OrthoGold100™ – OW100-US (K210451) Quick User Guide	Dokumentation_Titel_Revision (Code_Titel_Revision) MTS_OW100-US_IFU-Quick-Guide-OrthoGold100_E_210920	
		Ausgabedatum (Date of Release) (TT.MM.JJJJ) 20.09.2021	Seite (Page) 1 / 2

Attention: This document contains the OrthoGold100™ device Quick User Guide instruction for setup and operation.

1. Start the device.
 - 1.a Standby mode will be started after pressing the power button of the device.
 - 1.b Press the yellow button until the device starts running.
2. Attach water cartridge on the back of the device.
3. Plug applicator at the left-hand side of the device.
4. Press “**FILL**” button to fill the applicator head with water.
5. “**Setup**” individual treatment parameters by using the touch panel or the touch wheel (1.b).
6. Perform SoftWave therapy for the individual indication.
7. Press “**DRAIN**” button to drain water from applicator.
(Hold up the applicator head to drain all water from the applicator and its hose line.)
8. Press “**SETUP**” button to get back to setup screen mode.
9. Number of remaining SoftWaves of the applicator.
10. Press “**CARTRIDGE**” button to eject cartridge.
11. Press “**APPLICATOR**” button to remove applicator from device.
12. Press “**BACK**” button to return to main menu.
13. Press “**Yellow**” button until device shuts down (1.b.).
14. Press the power switch to turn off the model OW100-US SoftWave device (1.a.).
 - A. Displays the number of currently applied SoftWaves.
 - B. Displays the total energy of the currently applied SoftWaves.
 - C. Language setting of the menu.
 - D. Brightness setting of the display.
 - E. Configuration options for the unit of energy.
 - F. Adjustment of the date format.
 - G. Exact device designation:
 - Connected applicator type
 - Serial number
 - Software version
 - Number of applied SoftWaves of the device
 - H. Date and Time setting.

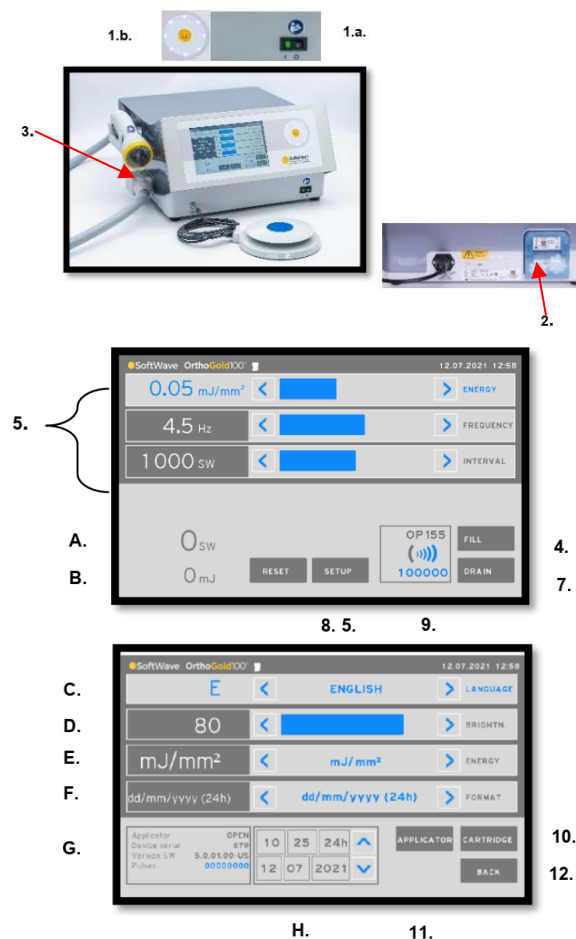


	IFU (Instruction for Use)		Dokumenennung_Titel_Revision (Code_Titel_Revision) MTS_OW100-US_IFU-Quick-Guide-OrthoGold100_E_210920	
	OrthoGold100™ – OW100-US (K210451) Quick User Guide		Ausgabedatum (Date of Release) (TT.MM.JJJJ) 20.09.2021	Seite (Page) 2 / 2

Operating Applicator

- Release of the predefined number of SoftWaves by
 - Permanently pressing the blue button or
 - Double-clicking the blue button.
- Replace membrane

When membrane (2.) needs to be replaced

 - Screw off the membrane ring manually.
 - Place a new membrane and retighten the ring.
- Do not bend the hose of the applicator to avoid damages.

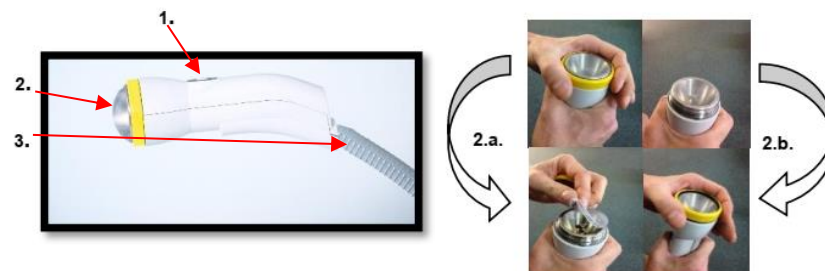

The OrthoGold100™ device may display some coded messages:

I - INFORMATION



I 714
(194 → 07141)

Please wait - adjustment and release of applicator in process.

24.05.2019
16.22.25

OK

1/1
(Ref. 2)

Info messages are specified by a text, a code I and a three-digit number. Press "ok" to continue.

W - WARNING



W 601

Caution test Caution test Caution test

25.4.2000
10:43:44

1/1

Warning messages are specified by a text, a code W and a three-digit number. If not solved by pressing "ok", restart the device.

E - ERROR



E 113

Error test Error test Error test

25.4.2000
11:01:20

←

OK

→

2/3

Error messages are specified by a text, a code E and a three-digit number. If not solved by pressing "ok", restart the device and call service if the error reappears.